


Combination Treatment for Acne with Cellec & IntraGen

Jeisys Medical Inc.


Acne Combination Treatment

- IntraGen → Cellec → IntraGen → Cellec → ...
- Weekly, 12 sessions, 3 Mo / IntraGen 6 sessions + Cellec 6 sessions

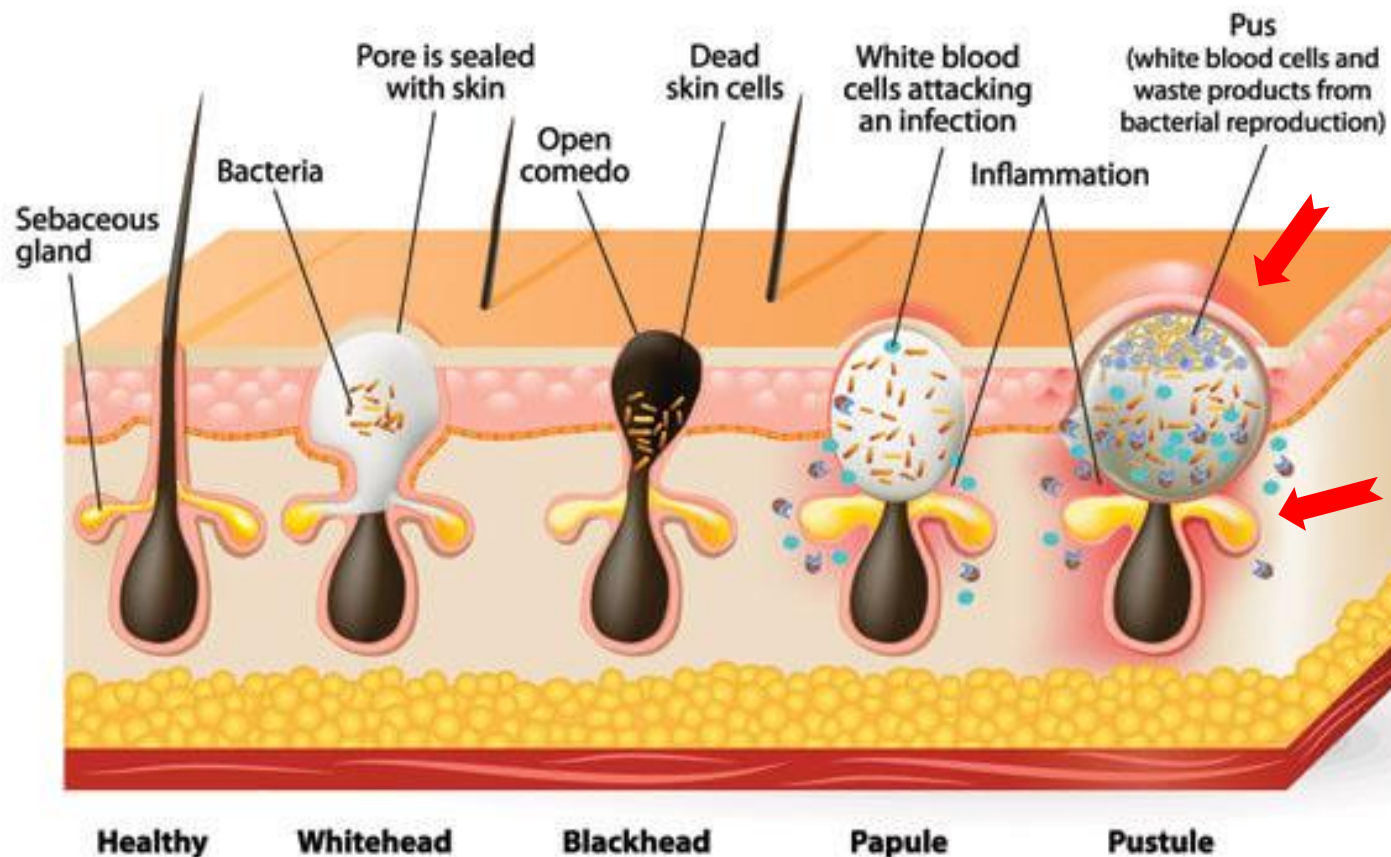

Acne Treatment with IntraGen

- IntraGen → Cellec → IntraGen → Cellec → ...
- Weekly, 12 sessions, 3 Mo / IntraGen 6 sessions + Cellec 6 sessions
- Reduction for sebum production

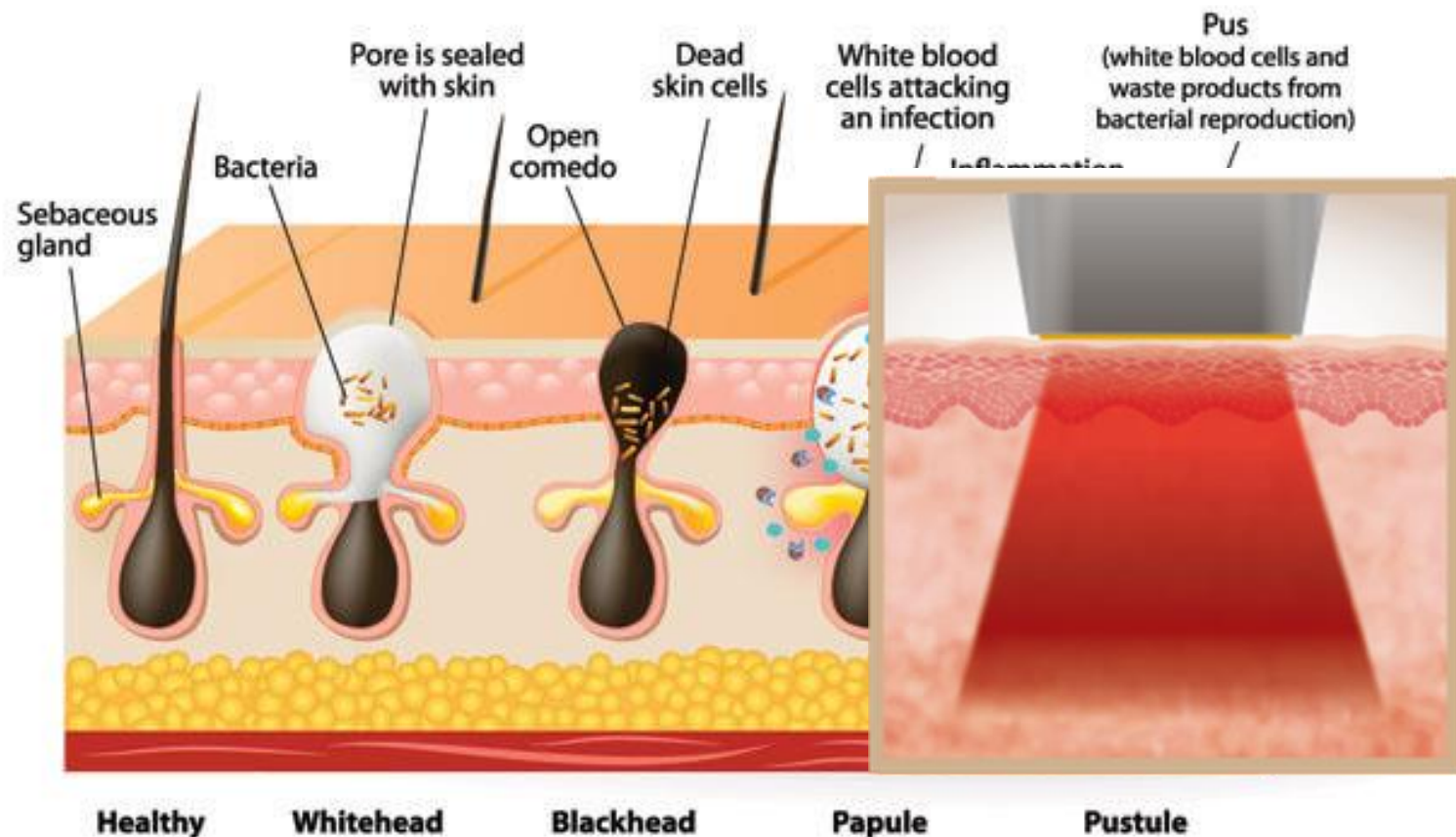


IntraGen Parameter for Acne

Area	Mode	Energy	Pass
Forehead, Nose, Philtrum	Stamp mode	Lv. 3	2 passes
Cheeks		Lv. 4	2 passes
Forehead, Nose, Philtrum	Sliding mode	Lv. 4	2 passes
Cheeks		Lv. 6	2 passes

- Start with IntraGen
- Every 2 weeks, 6 sessions
- Combine with Cellec 6 sessions
- 12 sessions, 3 Mo (total)

Acne with Cellec

- For pain care, oral treatment with Acetaminophen (Paracetamol), 30 min before treatment

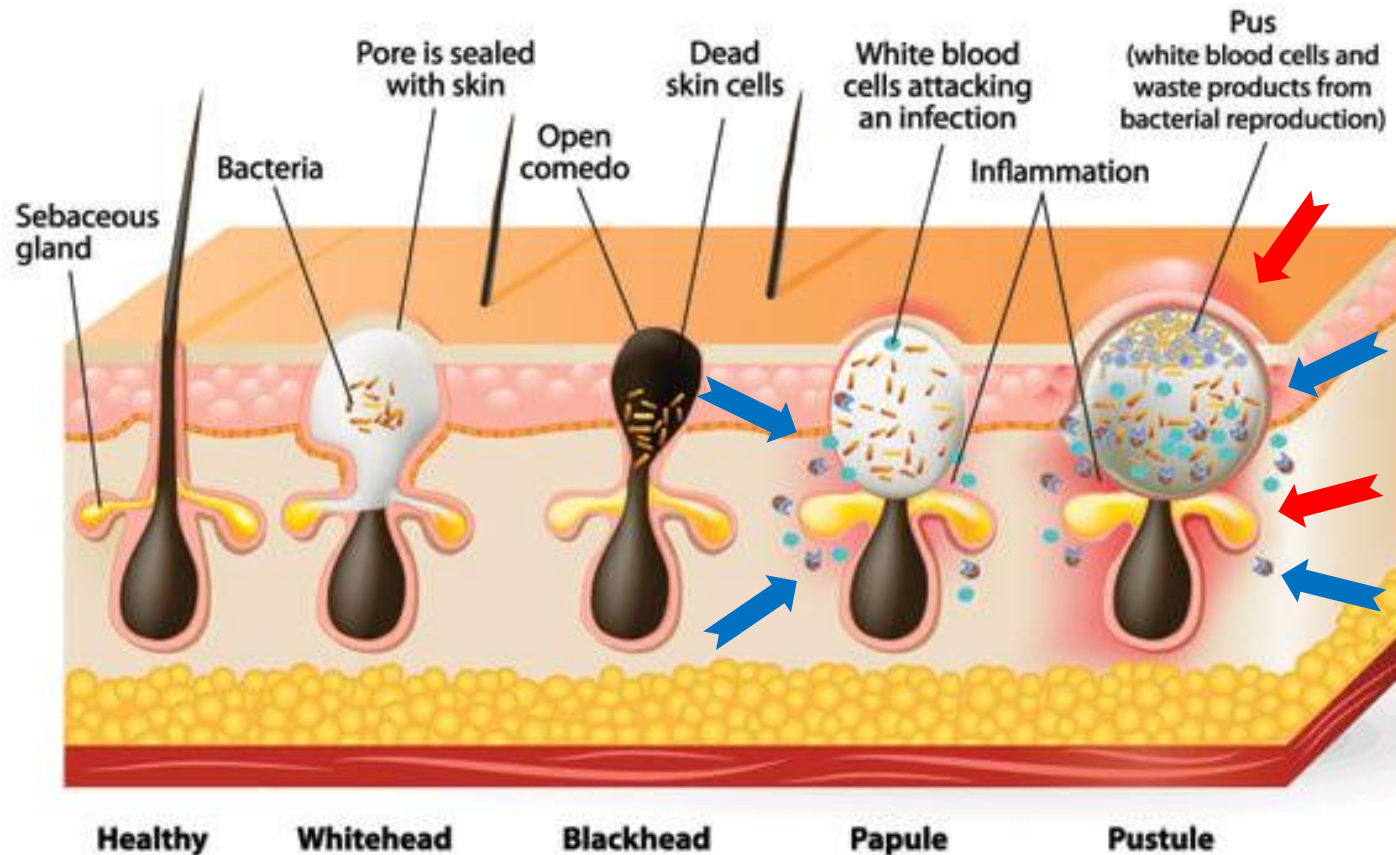

Cellec Parameter for Acne

	I ~ II	III	IV	V
Filter	420(s)	420(s)	420(s)	420(s)
Pulse Duration	2.4/4/4	2.4/5.0	2.8/5.4	2.8/5.4
Pulse Delay	20	20	30	40
Fluence(J/cm2)	12	11	10	9

- Apply Gel+Water mixture (1:1), spread with brush, gentle movement with Cellec tips
- Start with IntraGen
- Every 2 weeks, 6 sessions
- Combine with IntraGen 6 sessions
- 12 sessions, 3 Mo (total)