

---

*All New*  
**INTRAcel™**

Parameter Guide

v 1.1

---

# ACNE TREATMENT PARAMETER

## PARAMETER

	Mode	Depth[mm]	Level	Pass	Overlap[%]
Full Face	Bipolar	0.8	2~3	1	50%
Lesion	Monopolar	1.5	6~7	1	X

Treatment period may vary by ACNE condition(One treatment in general)

## DETAIL PARAMETER

### Level of pain

PAIN I Minor feeling (Need anesthetic cream)  
 PAIN II Minimally painful (Need anesthetic cream)  
 PAIN III Average pain ((Need anesthesia cream + Nerve Block )

Pain	Part	Mode	Standard Skin				Thin Skin			
			Depth	Level	User Mode		Depth	Level	User Mode	
					W	ms			W	ms
I	Full Face	Bipolar	0.8	2	18	50	0.5	1	8	70
	Lesion	Monopolar	0.8	4	12	120	0.8	2	8	100
II	Full Face	Bipolar	0.8	3	23	50	0.8	2	15	60
	Lesion	Monopolar	1.5	5	12	140	0.8	4	10	140

① After treating full face with 0.8 mm / Bipolar / Short Pulse

② Second treatment with 1.5mm / Monopolar / Long Pulse(same method for lesion) .

Skin may have sensitive feeling during the summer time due to the humidity and hot weather.

30% lower energy for the treatment is recommended.

# ACNE SCAR TREATMENT PARAMETER

## PARAMETER

	Mode	Depth[mm]	Level	Pass	Overlap[%]
Full Face	Bipolar	0.8	2	1	50%
Lesion	Bipolar	1.5	3~4	1	X

**35 days interval**

## DETAIL PARAMETER

### Level of pain

PAIN I Minor feeling (Need anesthetic cream)  
 PAIN II Minimally painful (Need anesthetic cream)  
 PAIN III Average pain ((Need anesthesia cream + Nerve Block)

Pain	Part	Mode	Standard Skin				Thin Skin			
			Depth	Level	User Mode		Dapth	Level	User Mode	
					W	ms			W	ms
II	Full Face	Bipolar	0.8	3	15	70	0.8	2	12	70
	Lesion	Bipolar	1.5	4	18	80	0.8	3	15	80
III	Full Face	Bipolar	0.8~1.5	3	12	90	0.8	3	10	100
	Lesion	Bipolar	1.5~2.0	5	20	80	1.5	4	12	110

**Pain level III (Bipolar 1.5 mm + 2.0mm) is recommended if only treating lesion.**

## SUMMER PARAMETER

	Part	Mode	Standard Skin				Thin Skin			
			Depth	Level	User Mode		Depth	Level	User Mode	
					W	ms			W	ms
Summer	Full Face	Bipolar	0.8	2	12	80	0.8	1	8	70
	Lesion	Bipolar	1.5	3	12	100	1.5	2	10	90

**Skin may have sensitive feeling during the summer time due to the humanity and hot weather.**

**30% lower energy for the treatment is recommended.**

# PORE TREATMENT PARAMETER

## PARAMETER

	Mode	Depth[mm]	Level	Pass	Overlap[%]
Full Face	Bipolar	0.8	2~3	1	50%
Lesion	Monopolar	1.5	4~5	1	X

First treatment: 35days Interval → Second treatment: 55 days Interval

## DETAIL PARAMETER

### Level of pain

PAIN I Minor feeling (Need anesthetic cream)  
 PAIN II Minimally painful (Need anesthetic cream)  
 PAIN III Average pain ((Need anesthesia cream + Nerve Block )

Pain	Part	Mode	Standard Skin				Thin Skin			
			Depth	Level	User Mode		Depth	Level	User Mode	
					W	ms			W	ms
I	Full Face	Bipolar	0.8	2	18	50	0.5	1	10	60
	Lesion	Monopolar	0.8	3	15	70	0.8	2	12	80
II	Full Face	Bipolar	0.8	3	18	60	0.8	2	12	70
	Lesion	Monopolar	1.5	4	22	60	1.5	3	15	80

# LIFTING TREATMENT PARAMETER

## PARAMETER

	Mode	Depth[mm]	Level	Pass	Overlap[%]
<b>1<sup>st</sup> step</b>	Monopolar	1.5~2.0	6~7	1	50%
<b>2<sup>nd</sup> step</b>	SRR Tip	-	3	1	X
<b>3<sup>st</sup> step</b>	Monopolar	0.8	4	1	50%
<b>4<sup>nd</sup> step</b>	SRR Tip	-	3	1	X

**1<sup>st</sup> step → (one week) → 2<sup>nd</sup> step → (three weeks) → 3<sup>st</sup> step → (one week) → 4<sup>nd</sup> step**

## DETAIL PARAMETER

### Level of pain

PAIN I Minor feeling (Need anesthetic cream)  
 PAIN II Minimally painful (Need anesthetic cream)  
 PAIN III Average pain ((Need anesthesia cream + Nerve Block )

Pain	Part	Mode	Standard Skin				Thin Skin			
			Depth	Level	User Mode		Depth	Level	User Mode	
					W	ms			W	ms
II	Lesion	Monopolar	0.8	4	10	130	0.8	3	10	110
III	Lesion	Monopolar	1.5~2.0	6~7	20	120	1.5	5	12	130

## SUMMER PARAMETER

	Part	Mode	Standard Skin				Thin Skin			
			Depth	Level	User Mode		Depth	Level	User Mode	
					W	ms			W	ms
Summer	Full Face	Bipolar	0.8	3	8	130	0.8	2	8	120
	Lesion	Monopolar	1.5~2.0	5	18	110	1.5	4	12	130

**Skin may have sensitive feeling during the summer time due to the humanity and hot weather.**

30% lower energy for the treatment is recommended.

# TIGHTENING TREATMENT PARAMETER

## PARAMETER

	Mode	Depth[mm]	Level	Pass	Overlap[%]
Full Face	Monopolar	0.8	2~3	1	50%
Lesion	Monopolar	1.5	6~7	1	X

1<sup>st</sup> treatment: 35 days Interval → 2<sup>nd</sup> treatment: 55 days Interval

## DETAIL PARAMETER

### Level of pain

PAIN I Minor feeling (Need anesthetic cream)  
 PAIN II Minimally painful (Need anesthetic cream)  
 PAIN III Average pain ((Need anesthesia cream + Nerve Block )

Pain	Part	Mode	Standard Skin				Thin Skin			
			Depth	Level	User Mode		Depth	Level	User Mode	
					W	ms			W	ms
I	Full Face	Monopolar	0.8	3	8	150	0.5	1~2	8	60
	Lesion	Monopolar	0.8	5	15	110	0.8	4	12	120
II	Full Face	Monopolar	0.8	4	10	140	0.8	3	10	120
	Lesion	Monopolar	1.5	6~7	18	130	1.5	5	15	130

## SUMMER PARAMETER

	Part	Mode	Standard Skin				Thin Skin			
			Depth	Level	User Mode		Depth	Level	User Mode	
					W	ms			W	ms
Summer	Full Face	Monopolar	0.8	2	10	90	0.8	2	10	90
	Lesion	Monopolar	1.5	4	15	100	0.8	3	10	120

Skin may have sensitive feeling during the summer time due to the humidity and hot weather.

30% lower energy for the treatment is recommended.

# STRIAE DISTENSAE TREATMENT PARAMETER

## PARAMETER

	Mode	Depth[mm]	Level	Pass	Overlap[%]
Lesion	Bipolar	1.5~2.0	2~3	3~4	30%

One month interval is required and minimum of 5 treatment is recommended.

## DETAIL PARAMETER

### Level of pain

PAIN I Minor feeling (Need anesthetic cream)  
 PAIN II Minimally painful (Need anesthetic cream)  
 PAIN III Average pain ((Need anesthesia cream + Nerve Block )

Pain	Part	Mode	Standard Skin				Thin Skin			
			Depth	Level	User Mode		Depth	Level	User Mode	
					W	ms			W	ms
I	Lesion	Bipolar	1.5~2.0	2~3	12	90	Equivalent as Standard Skin			

Since stretch marks are usually treated on ab-do-men or calf, maximum pressure on treating area is recommended.